



Scan QR code for more information and instructional video

MedBuddy®

Instructions

MedBuddy.org

MedBuddy®


- 1** Select the appropriate clip for the syringe size.




White **small** clip Black **large** clip

Please choose a clip that results in a snug fit between clip and syringe, allowing no movement.


- 2** Squeeze and insert clip into slot underneath toy.

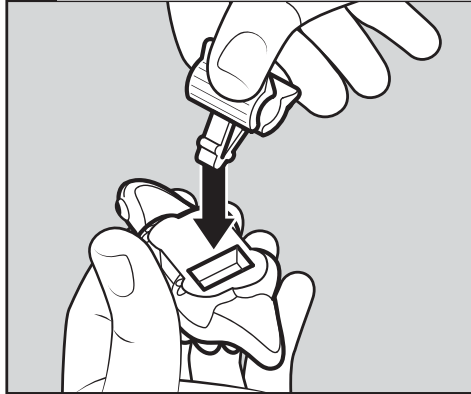


- 3** Snap toy and clip assembly onto syringe or vial. Ensure a snug fit.

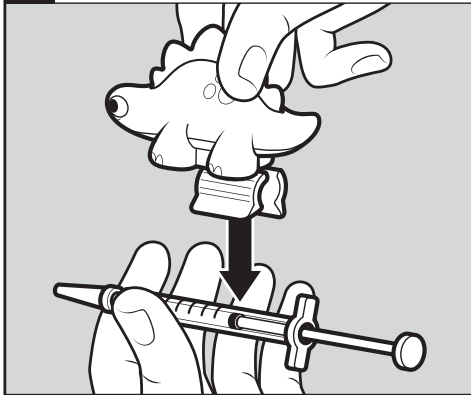

- 4** Use this completed assembly to administer injection.

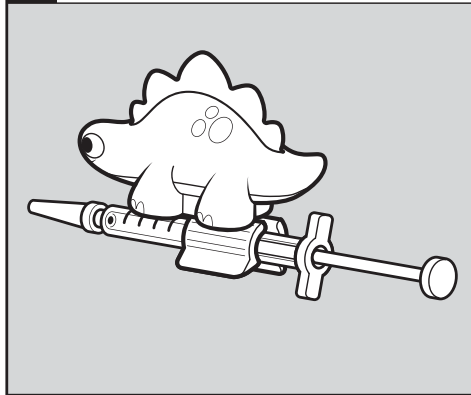

- 5** After injection, remove the toy and clip assembly from syringe.

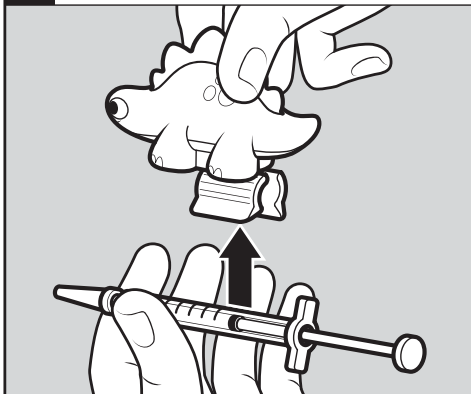

- 6** Squeeze and remove clip from toy. **Discard the clip and give only the toy to the child.**

